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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION

The upper arm blood pressure monitor is used for non-invasive measurement and monitoring of blood pressure in adults. It enables easy and quick blood pressure measurement, as well as saving the measured values and displaying them in the form of a graph. The user is warned about the occurrence of heart rhythm disturbances. The measured values are classified according to the guidelines of the World Health Organization and evaluated in graphic form.

The device is intended for home use only and is not intended for medical or commercial use and is not a substitute for medical therapy. The device is intended for use inside dry rooms. It is not intended for use in medical or therapeutic settings or for commercial use. The product is not a medical device.

For safety and CE certification reasons, the product may not be rebuilt or modified in any way. If the product is used for purposes other than those previously described, the product may be damaged. Improper use may also cause hazards such as short circuits, fire, electric shock, etc.

DEVICE DESCRIPTION (PICTURE A)

1. Air hose
2. Blood pressure monitor
3. Display
4. Cuff
5. Air connection
6. Function buttons
7. USB charging port

DISPLAY DESCRIPTION (PICTURE B)

1. User Group
2. Low battery symbol
3. Date and time
4. Diastolic pressure
5. Systolic blood pressure
6. Heart rate monitor
7. Heartbeat Symbol
8. Arrhythmia
9. The number of the saved measurement
10. Unit of measurement
11. Voice On/Off
12. Blood pressure classification bar
13. Memory Value Query
14. Settings
15. Device switch
16. Memory

BEFORE USE

Do not attempt to make a diagnosis based on measured values. Leave the interpretation of blood pressure readings to a doctor or other medical professional who is familiar with your condition. If you take medication regularly, ask your doctor what time of day is best for you to measure your blood pressure. Never change your prescribed medication without consulting your doctor.

People with irregular or unstable peripheral circulation due to diabetes, liver problems or stiff arteries may experience differences in results when measuring blood pressure at the wrist and at the upper arm.

The device is intended for use by adults only. Consult your doctor for blood pressure measurements in children and adolescents. Never use the device near devices such as TVs, microwaves, X-ray machines or mobile phones. These may cause incorrect measurements.

If the cuff is too tight or you feel discomfort, immediately press the switch to release the pressure.

STARTING THE DEVICE

- Note: Packaging material must be completely removed from the product.
- Make sure that the device components are properly assembled.
- Insert the batteries as marked, paying attention to the correct polarity (+ and -).

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

Inserting / Replacing the battery

- Open the battery compartment cover and remove it.
- Remove the used batteries. Insert new batteries into the battery compartment.
- Pay attention to the orientation of the poles (+ and -).
- Close the battery compartment cover.
- Do not use rechargeable batteries.

Connecting to USB Power

- Remove the batteries from the battery compartment before establishing a USB connection.
- Connect the USB cable to a USB power source.
- Connect the DC plug of the USB cable to the power socket on the product.


Selecting the electrical power supply

If the DC plug is not connected to the product, power is supplied by batteries.

Battery replacement instructions:


- Please insert the batteries correctly, paying attention to the +/- polarity.
- Only use one type of battery.
- Always replace the entire set of batteries.
- Exhausted batteries must be removed from the device and never thrown into fire.
- Never attempt to charge regular batteries.
- Store unused batteries in their packaging away from metal objects.
- Do not leave children unsupervised.

BLOOD PRESSURE MEASUREMENT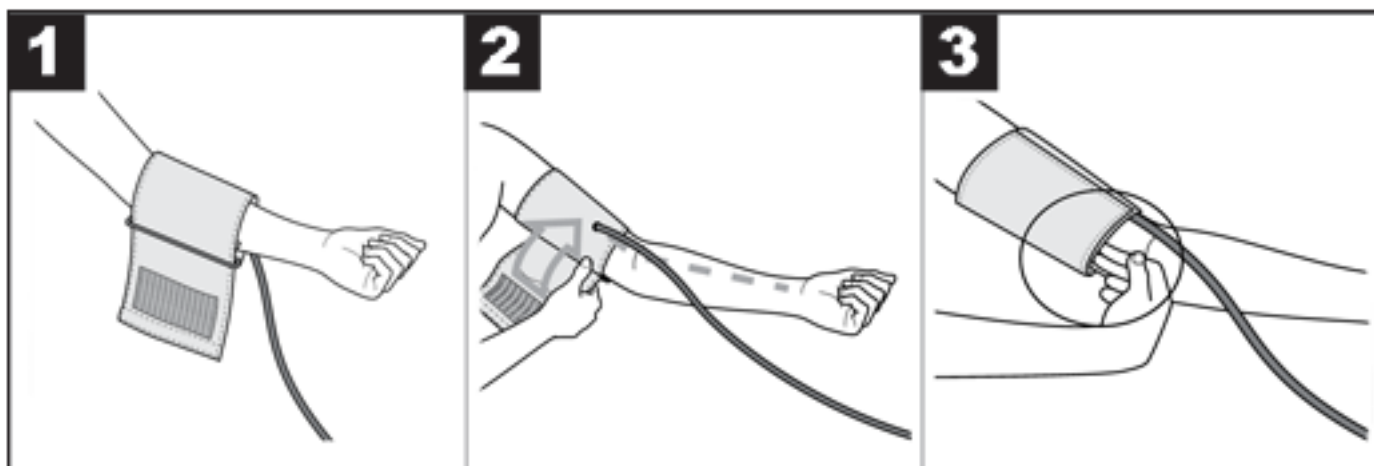

- NOTE! The device can only be used with the original cuff. The cuff is suitable for an arm circumference of 22 to 32 cm
- Place the cuff on the bare left arm. The blood circulation in the arm must not be restricted by tight clothing or the like.
- Place the cuff on the arm so that the lower edge is more than 2-3 cm above the antecubital fossa and above the artery. The cuff tube points to the center of the palm.
- Pull the free end of the cuff around your arm, but not too tightly, and fasten it with Velcro. The cuff should be tight enough to allow two fingers to fit under the cuff.
- Insert the cuff hose with cuff plug into the device connector.

CORRECT BODY POSITION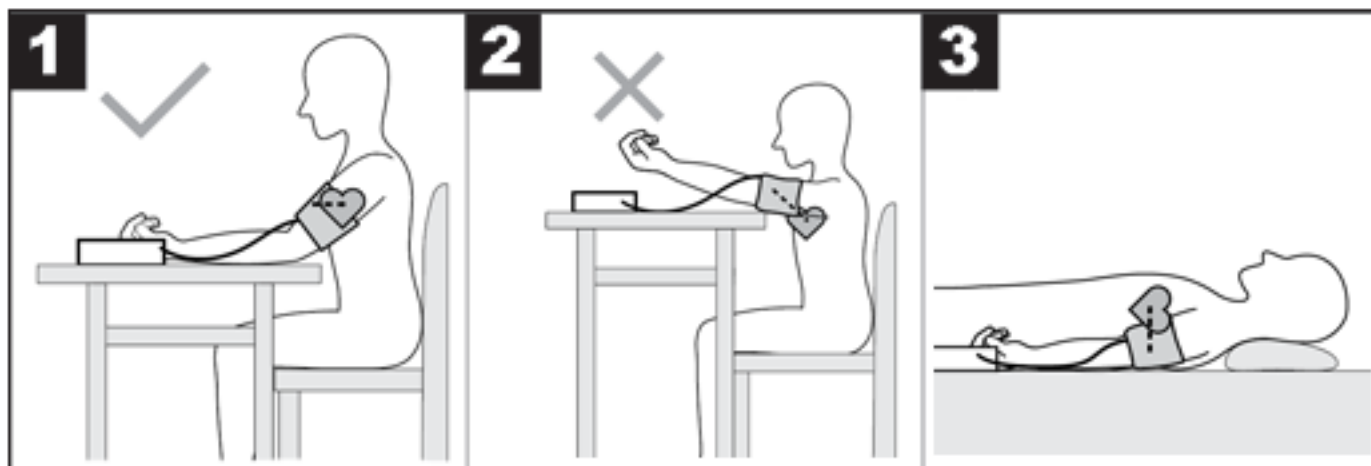

- Before taking a measurement, rest for about 5 minutes. Otherwise, the measurement may be inaccurate.
- Blood pressure can be measured in a sitting or lying position. Sit comfortably for blood pressure measurement. Support your back and arms. Do not cross your legs. Rest your feet flat on the floor. The arm should be supported and bent. Make sure that the cuff is always at the level of the heart. Otherwise, there may be large measurement inaccuracies. The arm and hand should be relaxed.
- Do not move or talk during the measurement.

BLOOD PRESSURE MEASUREMENT

- Place the blood pressure monitor on your arm according to the correct use.
- Turn on the device using the On/OFF button and the device will automatically start measuring.
- Do not move or speak during the measurement.
- After the voice transmission and after the display check, during which all the digits are displayed (if it is switched on), the device starts to pump air automatically to measure the blood pressure. The device measures the pressure already during the pumping in order to estimate the required inflation pressure. If this pressure is insufficient, the cuff is automatically pumped up.
- The air in the cuff is then slowly released and once a pulse is detected, the pulse symbol will flash.
- The whole process takes about 35 to 55 seconds. Depending on the actual blood pressure situation, the test time will change.
- Systolic, diastolic and pulse rate measurement results are displayed.

MEMORY DISPLAY

When the device is turned off, press the M button on the device housing to display the saved measurements for the user. To view the next saved measurement, press the M or S button on the device housing.

To delete all measurements for a given user, hold down the M button and when "DL" appears on the display, press the S button.

DEVICE SETTINGS

To access the device settings when it is turned off, hold down the S button on the device casing. Change the setting by pressing the M button.

- User Choice: 1 or 2
- Year
- Date (month/day of month)
- Time
- Turn on/off voice prompts
- Set the unit of measurement Kpa (the device works on mmHg by default)

INTERPRETATION OF RESULTS

Cardiac arrhythmias:

During the measurement, the device can detect any heart rhythm disturbances. The user is informed about this after the measurement is completed by the symbol "IHB" on the display. This may be a symptom of cardiac arrhythmia. Arrhythmia is a disease that causes heart rhythm disturbances due to errors in the bioelectric system that controls the heartbeat. Symptoms of this disease include missing or premature heartbeats, as well as a heart rate that is too slow or too fast. Causes include heart disease, old age, predispositions resulting from body structure, excessive consumption of stimulants, stress or too little sleep. Arrhythmia can only be diagnosed by a doctor after conducting an appropriate examination. If the symbol "IHB" appears on the display after the measurement is completed, the measurement must be repeated. Before starting the measurement, rest for at least 5 minutes and do not talk or move during the measurement. If the symbol "IHB" appears frequently, contact your doctor. Self-diagnosis and treatment based on measured values may be hazardous to your health. It is essential to follow the doctor's recommendations.

Classification according to the World Health Organization:

According to the guidelines/definitions of the World Health Organization and the current state of knowledge, the measurement results can be classified and interpreted according to the following table:

Pressure value range	Systolic blood pressure (in mmHg)	Diastolic pressure (in mmHg)	What to do
Level 3: High blood pressure	≥ 180	≥ 110	Go to the doctor
Level 2: moderate hypertension	160-179	100-109	Go to the doctor
Level 1: mild hypertension	140-159	90-99	Regular medical check-up
Normal high	130-139	85-89	Regular medical check-up
Normal	120-129	80-84	Self-control
Optimal	< 120	< 80	Self-control

The bar graph on the display and the scale on the device inform you about the range in which the measured pressure falls.

If the systolic and diastolic blood pressure values fall within two different World Health Organization ranges (e.g. systolic blood pressure in the "Normal High" range and diastolic blood pressure in the "Normal" range), the higher range is always displayed - in this example, the "Normal High" range.

ERROR MESSAGES

If errors occur, the display shows E_. Error messages can appear if:

- Er: The measured pressure was below 20mmHg. Please repeat the measurement.
- Er: The pressure cannot be measured due to signal noise. Please repeat the measurement.
- Er: The measured pressure was higher than 300mmHg. Please repeat the measurement.
- Er: The cuff was not wrapped correctly, or the user was talking or moving during the measurement. Please repeat the measurement.
- ErH HI: The inflation pressure is higher than 300mmHg. Please repeat the measurement.
- If no measurement occurs, replace the batteries.

TECHNICAL DATA

- Product material: ABS material
- LED display
- Measurement method: oscillometric, non-invasive measurement of blood pressure on the upper arm
- Arm circumference: 22 cm ~ 32 cm
- Pressure accuracy: ± 3 mmHg
- Heart rate accuracy is $\pm 5\%$
- Static pressure: 0-299 mmHg
- Heart rate: 40 ~ 199 times/min
- Automatic inflation and deflation
- Automatic shutdown
- Switching user 1/2
- Power supply: 4 x AAA battery or USB DC 5V

APPLICATION DIRECTIONS

- Always measure your blood pressure at the same time of day so that the measured values are comparable.
- Rest for approximately 5 minutes before each measurement.
- If you want to take several measurements on one person, keep a 5-minute break between measurements.
- Do not eat, drink, smoke or exercise for at least 30 minutes before taking a measurement.
- Repeat the measurement if the measured value is questionable.
- The measured values are for information purposes only, blood pressure measurement does not replace a medical examination! After measuring blood pressure, you should seek medical advice. Under no circumstances should you make medical decisions on your own (e.g. regarding the use of medications and their dosage) based on the measurement!
- Do not use the blood pressure monitor on newborns, during pregnancy or on women suffering from toxemia of pregnancy.
- In the case of problems with blood supply to the arm due to chronic or advanced vascular disease (including narrowing of blood vessels), the accuracy of the wrist blood pressure monitor is limited. In such a case, an upper arm blood pressure monitor should be used.
- Circulatory system diseases can cause measurement errors or impair measurement accuracy. This also applies to very low blood pressure, diabetes, heart rhythm and circulation disorders, as well as chills and tremors.
- The blood pressure monitor must not be used together with high-frequency surgical equipment.
- The device should only be used on people with the specified wrist circumference.
- While inflating the device, the functionality of the affected limb may be impaired.
- Blood circulation must not be disturbed by taking blood pressure for too long. If the device malfunctions, remove the cuff from the arm.
- Avoid holding pressure in the cuff and taking frequent measurements. The resulting disruption of blood flow may cause damage to health.
- The cuff should not be placed on the arm where arteries or veins are being treated, e.g. angioplasty/vascular therapy or arteriovenous (AV) fistula.
- Do not apply the cuff to persons who have had a mastectomy.
- Do not place the cuff over wounds as further injury may occur.

- If no button is pressed within 1 minute, the blood pressure monitor will automatically turn off to save battery life.
- The device is intended only for use for the purpose specified in this user manual. The manufacturer is not responsible for damage resulting from improper use of the device.

STORAGE AND USE INSTRUCTIONS

The blood pressure monitor is made of precision electronic components. Careful handling of the device is closely related to the accuracy of measurements and the length of the service life.

- Protect the device from shocks, moisture, contamination, large temperature fluctuations and direct sunlight.
- Do not drop the device.
- Do not use the blood pressure monitor near strong electromagnetic fields, radio equipment or mobile phones.
- Do not press buttons before applying the cuff.
- Do not place heavy objects on the device.

SAFETY INSTRUCTIONS

- Always follow your doctor's instructions. Do not attempt to diagnose or treat any conditions yourself. This may be dangerous. Do not use the device for first aid, health assessment or continuous monitoring.
- This product may be used by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge if they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- The device is not intended for use by persons (children or adults) with any form of functional disabilities or persons lacking sufficient experience or knowledge in the use of the device, unless they receive instructions regarding the use of the device from a person responsible for their safety.
- Do not use a mobile phone near the device. Do not use the device in or near a magnetic field.
- Under no circumstances may you open or repair the device, as otherwise its proper functioning cannot be guaranteed.
- Never immerse the device in water.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- The product is not intended for use by children.
- Make sure that packaging materials are not left unattended. Children may start playing with them, which is dangerous.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - does not function properly, - has been stored for an extended period under unfavourable conditions or - has been excessively stressed during transport.
- It is forbidden to use the product if any of its parts are damaged. In the event of damage to the cable, it is forbidden to make repairs yourself.
- Do not disassemble the device yourself.
- Protect the product from moisture.
- Make sure not to damage the USB cable with sharp edges or hot objects. Unwind the USB cable completely before use.
- Turn off the product immediately and remove the cable from the product if it smells burning or smoke is visible. Have the product checked by a qualified technician before using it again.
- If a USB power supply is used, the socket must always be easily accessible so that the USB power supply can be easily removed from the socket in an emergency. Please also observe the manufacturer's operating instructions.
- Before each charging process, check the USB cable and device for any damage. Never use or charge the device if you find any damage.
- Before use, make sure that the existing mains voltage complies with the required operating voltage of the device.
- Always use the product as intended.

BATTERIES/ACCUMULATORS SAFETY INSTRUCTIONS

- Batteries/rechargeable batteries should be kept out of reach of children. If swallowed, consult a doctor immediately!
- Disposable batteries must not be recharged. Batteries/accumulators must not be short-circuited and/or opened. This may cause overheating, fire or explosion.
- Never throw batteries/accumulators into fire or water.
- Batteries/accumulators should never be subjected to mechanical stress.


- Risk of acid leakage from batteries/accumulators.
- Avoid extreme conditions and temperatures that may affect batteries / accumulators, e.g. radiators / direct sunlight.
- If batteries/accumulators leak, avoid contact of skin, eyes and mucous membranes with chemicals! Immediately rinse affected areas with clean water and seek medical attention!
- Leaking or damaged batteries/rechargeable batteries can cause chemical burns when in contact with skin. Therefore, suitable protective gloves should be worn in such cases.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.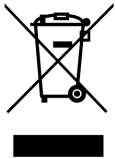

DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT

Used electrical and electronic products, due to environmental protection, must not be thrown into household waste with regular municipal waste, but disposed of properly. Information on collection points and their opening hours is provided by the appropriate office.

This product complies with the requirements of the relevant European and national directives that apply to it. The product meets European and national requirements on the safety of devices and products.

This product complies with the requirements of the relevant European and national directives that apply to it, RoHS.

Improper disposal of batteries/accumulators poses a risk to the environment!

Batteries/accumulators should not be disposed of with household waste. They may contain harmful heavy metals and should be treated as special waste. The chemical symbols for heavy metals are as follows: Cd = cadmium, Hg = mercury, Pb = lead.

Therefore, used batteries / accumulators should be taken to municipal hazardous waste collection points.

The built-in battery must not be disassembled for disposal. The product must be handed over in its entirety to a collection point for used electronics.

We reserve the right to make changes to the text, design and technical data of the product without notice.

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