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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read the following instructions to ensure proper use of the product.

Please keep this manual for future reference and follow its recommendations, as failure to follow its instructions may pose a threat to life or health.

APPLICATION AND DESCRIPTION OF THE DEVICE

The knee band with heating function is an innovative device designed to relieve pain and support the regeneration of the knee joint . Thanks to the use of heat technology , it effectively relaxes muscles, reduces stiffness and supports the treatment of joint injuries and diseases.

The device is intended for home use only and is not intended for medical or commercial use and is not a substitute for medical therapy. The device is intended for indoor use in dry rooms. It is not intended for medical or therapeutic use or for commercial use.

For safety and CE certification reasons, the product may not be rebuilt or modified in any way. If the product is used for purposes other than those previously described, the product may be damaged. Improper use may also cause hazards such as short circuits, fire, electric shock, etc.

WARNING

- Danger of improper use!
- Using the device for purposes other than its intended purpose and/or using the device in any other way may involve various risks.
- The device must only be used for its intended purpose.
- The procedures described in this instruction manual must be followed.

Note : It is recommended to consult a doctor before using the brace, especially if you have knee diseases, inflammation, joint injuries or skin problems in the knee area.

COMMISSIONING/INSTALLATION INSTRUCTIONS

- Lay your band out on a flat surface and make sure it is connected to power if your device requires a wired connection or charging.
- Make sure the surface of the band is clean and dry before placing it on your knee.
- Place the band on your knee, making sure the center of the device is directly above your knee joint.
- Secure the brace by fastening the Velcro straps so that it fits snugly around the knee but does not cause discomfort or pressure. The brace should fit comfortably for optimal effectiveness.
- Find the Power button on the control panel and hold it for 2-3 seconds to start the device function.
- It is recommended to use the band for a maximum of 15-20 minutes per session, unless the manufacturer's instructions indicate otherwise.
- Avoid prolonged use to prevent skin overheating.
- When you have finished using the device, press the Power button on the control panel to turn it off.
- Wait a few minutes for the device to cool down before removing it from your knee.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or if you notice damage due to faulty packaging or transport, please contact the service hotline.

TECHNICAL DATA

- Material: 100% polyester
- Fastened with Velcro
- One size fits all
- 3 heating levels
- It has a pocket for a cooling insert
- Power supply: USB DC 5V

SAFETY INSTRUCTIONS

- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Never immerse the device in water.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- The product is not intended for use by children.
- Children must not play with the appliance.
- Cleaning and user maintenance shall not be performed by children without supervision.

- The product itself and its accessories are not toys, please keep them out of the reach of children to avoid danger.
- Make sure that packaging materials are not left unattended. Children may start playing with them, which is dangerous.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - does not function properly, - has been stored for an extended period under unfavourable conditions or - has been excessively stressed during transport.
- It is forbidden to use the product if any of its parts are damaged. In the event of damage to the cable, it is forbidden to make repairs yourself.
- Do not use if the power cord or power adapter is damaged or if it is not properly plugged into the outlet.
- Do not disassemble the device yourself.
- Do not hit or drop this product. Stop using it if it is damaged in a fall or other mechanical damage.
- Store the device in a cool and dry place.
- Before using the device, make sure that the mains voltage matches the required operating voltage of the device.
- Always use the product as intended.

WARNING! INJURY RISK!

- This device must not be used to massage infants, small children, helpless persons or animals.
- Never use this device if you experience dislocation or pain in the area being massaged.
- Stop the massage immediately if it becomes uncomfortable or causes pain.
- The device has a hot surface during operation. People who are not sensitive to heat should exercise caution when using the device.
- Do not use the device on swollen, burned, irritated, diseased or injured areas of the skin or body.
- If you have any doubts regarding use, please consult your doctor.
- The device must not be used on the face (e.g. eyes), larynx, feet or other sensitive parts of the body.
- Do not cover the device (pillow, blanket, etc.) during use. Do not use the device near gasoline or other flammable materials.

WARNING – MATERIAL DAMAGE!

- Use the appliance indoors only.
- Only use the device with the supplied mains adapter.

TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING

The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center.

Used packaging material should be delivered to a waste disposal point designated by local authorities. Information on the possibilities of recycling the used product is provided by the municipal or city office.

DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT

Used electrical and electronic products, due to environmental protection, must not be thrown into household waste with regular municipal waste, but disposed of properly. Information on collection points and their opening hours is provided by the relevant office.

This product complies with the requirements of the relevant European and national directives that apply to it. The product meets European and national requirements on the safety of devices and products.

This product complies with the requirements of the relevant European and national directives that apply to it, RoHS.

Warning of electric shock! Danger to life!

We reserve the right to make changes to the text, design and technical data of the product without notice.

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