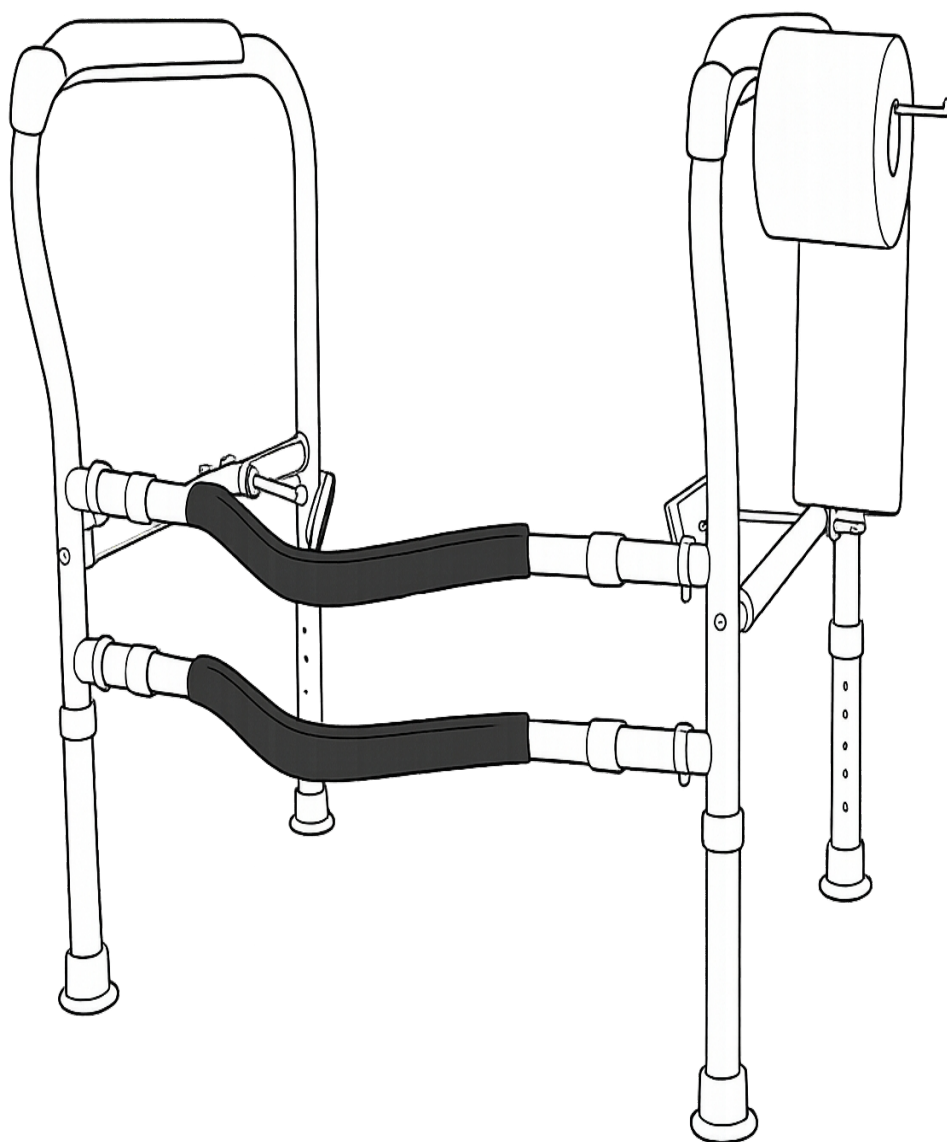
INSTRUKCJA

OBSŁUGI

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Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read carefully with the instructions below for proper use of the product.

Please keep this manual for future reference and following its recommendations, because failure to follow it may pose a threat to life or health.

Introduction to the user manual

This user manual has been prepared to help you familiarize yourself with the proper and safe use of the device. It contains important instructions on operation, maintenance, and safety rules, which, if followed, will ensure the smooth operation of the device and extend its service life.

Before first use, please read the instructions carefully. The information contained herein will also help you properly store the device, especially if it will not be used for an extended period.

Following the recommendations below will allow you to enjoy trouble-free and safe operation of the product.

ATTENTION!

Using the device in a manner inconsistent with the information contained in the manual may result in the risk of bodily injury.

Application and description of the device

The toilet rail is designed to provide stable and safe support while using the toilet. It is especially recommended for the elderly, those with limited mobility, post-operative patients, and those undergoing rehabilitation .

WARNING!

- Danger of improper use!
- Using the device in a manner other than its intended purpose and/or using the device in any other way may involve various risks.
- The device should only be used for its intended purpose.
- Please follow the procedures described in this instruction manual.
- Keep the product away from children.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or damage due to faulty packaging or transport, please contact the service hotline.

Technical data of the device

Symbol	27421
EAN	5907451370568
Material	plastic + metal
Non-slip handles	Yes
Height adjustment	5 levels
Width adjustment	2 levels
Toilet paper holder	Yes

Installation

PREPARATION FOR INSTALLATION

- Open the package and remove all components.
- Lay out the parts on a flat, stable surface.
- Assembly does not require tools – all elements are pressed together.

ASSEMBLY OF ELEMENTS

- Position the left and right side railing frames vertically:
 - the handles should be at the top,
 - rubber feet pointing down.
- Check that the rubber feet are properly seated on the ends of the tubes.
- Connect the side frames to the cross members:
 - insert the connecting elements into the slots in the side frames,
 - press them as far as they will go.
- Secure the connections with locking pins – the pin must “click”.

HEIGHT ADJUSTMENT

- There are adjustment holes on the handrail legs.
- Extend the lower legs to the desired height level.
- Insert the locking pins into the appropriate holes on both sides.
- Make sure that:
 - both sides are set at the same height,
 - the pins are completely locked.

HEIGHT ADJUSTMENT

- Extend or close the side rail frames to fit the width of the toilet.
- The handrail should enclose the toilet bowl and not rest directly on the ceramic.
- Lock your chosen setting using the latching mechanism.

Safety tips and rules

- Do not use a product that may be damaged or has sharp edges or other dangerous areas that could cause injury.
- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Always place the product on a stable and dry surface.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged product.
- Do not use the product near heat sources, open flames or in damp rooms.
- The product is not intended for use by children.
- Cleaning and user maintenance must not be performed by children without supervision.
- The product itself and its accessories are not toys and should be kept out of the reach of children to avoid danger.
- Keep packaging and all protective materials out of the reach of children as they may pose a risk of suffocation.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - is not functioning properly, - has been stored for an extended period under unfavorable conditions, or - has been subjected to excessive stress during transport.
- Do not use the product if any part is damaged. If the cord is damaged, do not attempt to repair it yourself.
- Do not hit or drop this product. Stop using it if it is damaged by a fall or other mechanical damage.
- The product should be stored in a cool and dry place.
- Only adjust the height or width when the handrail is not in use.
- If you notice any looseness, instability or other irregularities, discontinue use immediately.
- Do not rest your entire body weight on one side of the railing – the weight should be distributed evenly.
- Do not sit, stand or hang on the railing – the product is not designed to bear such a load.
- The product should always be used as intended.

Warning and information symbols

	<u>TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING</u> The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center. Used packaging material should be delivered to a designated waste disposal site designated by local authorities. For information on recycling options for used products, please contact your local municipality or city office.
---	---

We reserve the right to make changes to the text, design and product data without notice.

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