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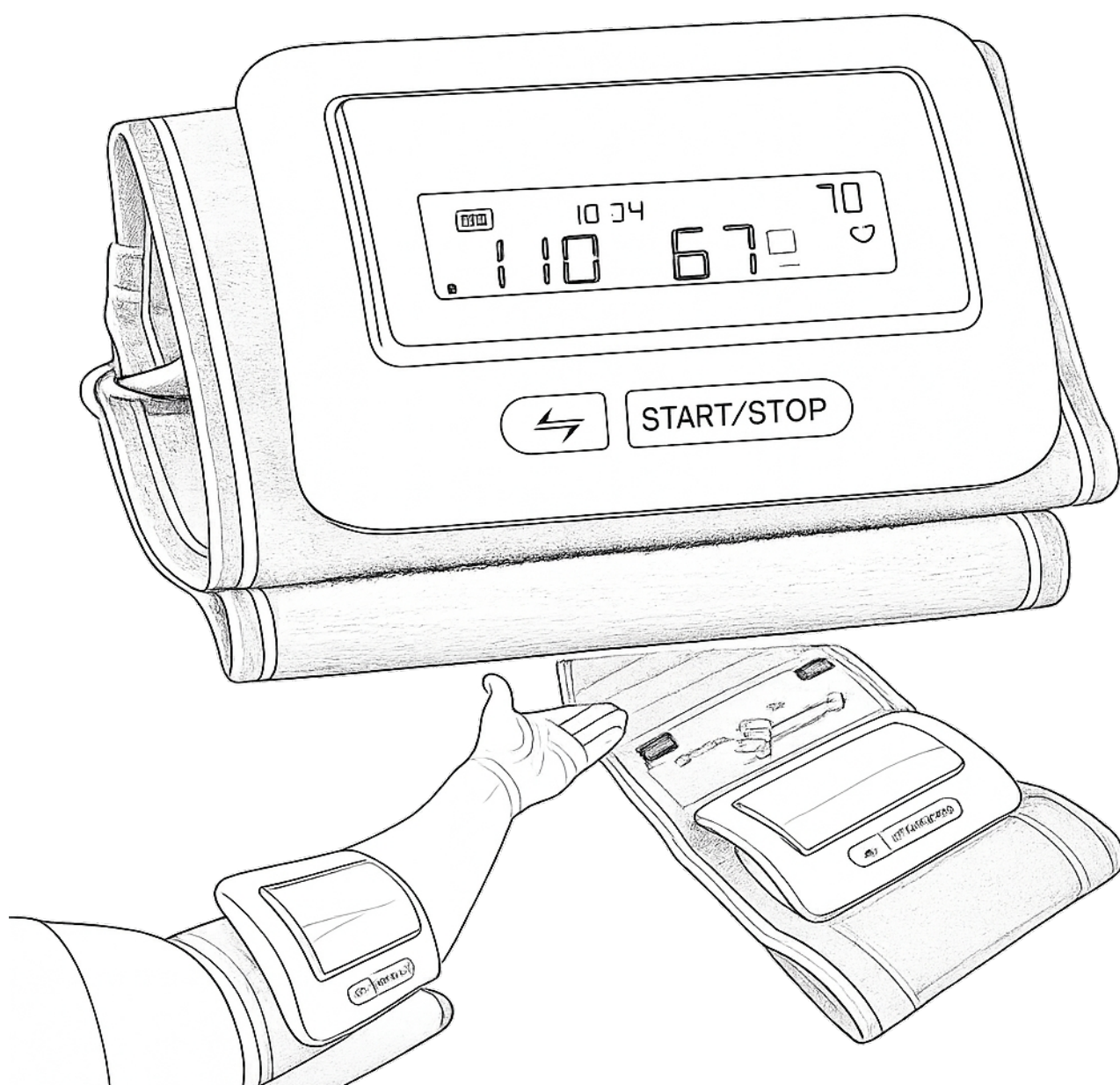
Dear Sir or Madam, thank you for purchasing our product!

Before using the product, please read carefully with the instructions below for proper use of the product.

Please keep this manual for future reference and following its recommendations, because failure to follow it may pose a threat to life or health.

Introduction to the user manual

This user manual has been prepared to help you familiarize yourself with the proper and safe use of the device. It contains important instructions on operation, maintenance, and safety rules, which, if followed, will ensure the smooth operation of the device and extend its service life.

Before first use, please read the instructions carefully. The information contained herein will also help you properly store the device, especially if it will not be used for an extended period.

Following the recommendations below will allow you to enjoy trouble-free and safe operation of the product.

ATTENTION!

Using the device in a manner inconsistent with the information contained in the manual may result in the risk of bodily injury.

Application and description of the device

An upper arm blood pressure monitor is a device used to measure blood pressure and pulse. It is intended for home use and allows for quick and convenient regular health monitoring. It is especially recommended for people with hypertension and those who care about preventative health care.

This device is intended for home use only and is not intended for medical or commercial use, and is not intended to replace medical therapy. The device is intended for indoor use in dry rooms. It is not intended for medical, therapeutic, or commercial use.

For safety and CE certification reasons, this product must not be rebuilt or modified in any way. Using the product for purposes other than those described may result in damage to the product. Improper use may also result in hazards such as short circuits, fire, electric shock, etc.

WARNING!

- Danger of improper use!
- Using the device in a manner other than its intended purpose and/or using the device in any other way may involve various risks.

- The device should only be used for its intended purpose.
- Please follow the procedures described in this instruction manual.
- Keep the product away from children.

TIP

- The device must be checked for completeness of delivery and any visible damage.
- In the event of an incomplete delivery or damage due to faulty packaging or transport, please contact the service hotline.

Technical data of the device

Symbol	27523
EAN	5907451373811
Measurement method	oscillometric
Power supply	4xAAA batteries
Arm circumference	22cm~32cm
Pressure	±3 mmHg (within ±0.4 kPa)
Heart rate accuracy	±5%
Static pressure	0-280 mmHg (0.0-37.33 KPa)
Pulse	40~199 times/min

Startup/Use

- Please remove the packaging materials and dispose of them properly and clean the product and accessories.
- Insert the batteries as marked, paying attention to the correct polarity (+ and -).
- Close the compartment and make sure the battery is seated correctly.
- Sit comfortably and rest for a few minutes before taking the measurement.
- Place the cuff on your upper arm (approximately 2–3 cm above the bend of your elbow).
- Make sure the cuff fits snugly, but not too tight.
- Rest your hand on a flat surface at heart level.
- Press the START/STOP button to start the measurement.
- Do not move or speak during the measurement.
- Once finished, read the result on the display.
- Save the result or use the device's memory function.
- Remove the cuff after completing the measurement.

Battery replacement instructions:

- Please insert the batteries correctly, paying attention to the +/- polarity.
- Only one type of battery should be used.
- Always replace the entire set of batteries.
- Exhausted batteries should be removed from the device and never thrown into fire.
- Never try to charge regular batteries.

-
- Store unused batteries in their packaging away from metal objects.
 - Do not leave children unsupervised.

Safety tips and rules

- This product may be used by children aged 8 years and over and by persons with reduced physical, sensory or mental capabilities or lack of experience or knowledge, provided they are supervised or have been instructed on the safe use of the product and understand the resulting risks.
- Never immerse the device in water.
- For cleaning, use a damp cloth or a mild detergent.
- Do not use a damaged device.
- The product is not intended for use by children.
- Make sure packaging materials are not left unattended. Children may play with them, which is dangerous.
- Protect the product from extreme temperatures, direct sunlight, strong vibrations, high humidity, moisture, flammable gases, vapors and solvents.
- Do not expose the product to mechanical stress.
- If safe operation is no longer possible, discontinue use and secure the product against further use. Safe operation is no longer possible if the product: - has been damaged, - is not functioning properly, - has been stored for an extended period under unfavorable conditions, or - has been subjected to excessive stress during transport.
- Do not use the product if any part is damaged. If the cord is damaged, do not attempt to repair it yourself.
- Do not disassemble the device yourself.
- Do not use old and new batteries, different brands or chemicals.
- Do not charge, damage or open the battery.
- Do not swallow the battery. If a battery is swallowed, consult a doctor immediately.
- Do not take the measurement immediately after physical exercise.
- Do not use the device if the cuff is damaged.
- Do not use the device on children without consulting a doctor.
- Do not place the cuff on an injured or sore arm.
- Don't take measurements too often – keep an interval of a few minutes.
- Do not modify the device or its components.
- The measurement results are indicative and do not replace a medical diagnosis.
- If you experience any disturbing results, consult your doctor.
- Do not use the device near strong electromagnetic fields.
- The product should always be used as intended.

Safety instructions for batteries/rechargeable batteries

- Keep batteries/rechargeable batteries out of the reach of children. If swallowed, seek medical attention immediately!
- Disposable batteries must not be recharged. Batteries/rechargeable batteries should not be short-circuited and/or opened. This may result in overheating, fire, or explosion.
- Never throw batteries/rechargeable batteries into fire or water.
- Batteries/rechargeable batteries should never be subjected to mechanical stress.
- Risk of acid leakage from batteries/accumulators.
- Avoid extreme conditions and temperatures that may affect batteries/accumulators, e.g. radiators/direct sunlight.
- If batteries/rechargeable batteries leak, avoid contact with skin, eyes, and mucous membranes! Immediately flush the affected areas with clean water and seek medical attention!
- Leaking or damaged batteries/rechargeable batteries can cause chemical burns when in contact with skin. Therefore, wear appropriate protective gloves.
- The product has a built-in battery that cannot be replaced by the user.

Warning and information symbols

	<u>TIPS AND INFORMATION ON THE MANAGEMENT OF USED PACKAGING</u> The packaging is made of environmentally friendly materials that can be disposed of at your local recycling center. Used packaging material should be delivered to a designated waste disposal site designated by local authorities. For information on recycling options for used products, please contact your local municipality or city office.
	<u>DISPOSAL OF USED ELECTRICAL AND ELECTRONIC EQUIPMENT</u> For environmental reasons, used electrical and electronic products should not be disposed of with regular household waste, but should be disposed of properly. Information on collection points and their opening hours can be obtained from the relevant authorities.
	This product complies with the requirements of the applicable European and national directives. The product meets European and national requirements for the safety of devices and products.
	This product complies with the requirements of the relevant European and national RoHS directives.
	<u>Improper disposal of batteries/accumulators poses a threat to the environment!</u> Batteries/accumulators should not be disposed of with household waste. They may contain harmful heavy metals and should be treated as special waste. The chemical symbols for heavy metals are: Cd = cadmium, Hg = mercury, Pb = lead. Therefore, used batteries/accumulators should be taken to municipal hazardous waste collection points. The built-in battery must not be disassembled for disposal. The product should be taken to a collection point for used electronics.

We reserve the right to make changes to the text, design and product data without notice.

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